

HOW TO HELP A VICTIM OF SUDDEN CARDIAC ARREST

SCA can happen anywhere, any time. When it does, the heart unexpectedly stops beating properly, and the victim's best chance of survival depends on immediate CPR and a rescuer with an AED.

Don't panic. You can help.

HOW YOU CAN HELP

Heart Saver CPR — 2020 AHA Guidelines

Check for Danger

Check Responsiveness

Call for Help
Call 911

Check for Breathing

Start CPR


Attach Defibrillator (AED)
as soon as possible and
follow prompts

HOW ZOLL CAN HELP

ZOLL® AEDs with Real-time CPR Feedback



Check Responsiveness
"Are you okay?"

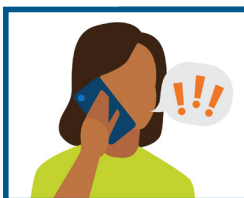

Call for Help
Call 911




"Attach AED pads to
patient's bare chest"




"Don't touch patient,
analyzing"



If shock is advised:
It will automatically be
delivered

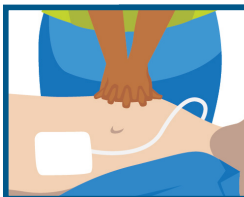

If no shock is advised and
after shock delivered:
"Start CPR"

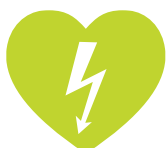

Not all AEDs are created equal. ZOLL AEDs provide support throughout the CPR process with real-time guidance and feedback on compression rate and depth. This helps responders deliver the best possible CPR to increase an SCA victim's chance of survival.